

ACCU-CHEK® Guide Link

User's Manual

Blood Glucose Meter

For use with compatible MiniMed™ Pump
with *Bluetooth®* wireless technology



Medtronic



Contents

Introduction	3
Chapter 1: Your New Meter	5
Chapter 2: Blood Glucose Tests.....	13
Chapter 3: Control Tests.....	27
Chapter 4: Meter Settings.....	33
Chapter 5: Review Your Data.....	55
Chapter 6: Wireless Communication and Meter Pairing.....	69
Chapter 7: Meter Maintenance and Troubleshooting	79
Chapter 8: Technical Information	91
Chapter 9: Guarantee.....	97
Index	99

The Accu-Chek Guide Link System

The Accu-Chek Guide test strips with the Accu-Chek Guide Link meter are intended to quantitatively measure glucose in fresh capillary whole blood from the finger, palm, forearm, and upper arm as an aid in monitoring the effectiveness of glucose control.

The Accu-Chek Guide test strips with the Accu-Chek Guide Link meter are intended for in vitro diagnostic self-testing by people with diabetes.

The Accu-Chek Guide test strips with the Accu-Chek Guide Link meter are intended for in vitro diagnostic use by healthcare professionals in clinical settings. Venous, arterial, and neonatal blood testing are limited to healthcare professional use.

This system is not for use in diagnosis of diabetes mellitus and not for testing neonate cord blood samples.

The Accu-Chek Guide Link Blood Glucose Monitoring System is intended to be used to wirelessly transmit glucose values to a compatible MiniMed™ Pump with Bluetooth® wireless technology through the use of Bluetooth low energy communication.

Suitable for self-testing.

The system includes:

Accu-Chek Guide Link meter with batteries, Accu-Chek Guide test strips*, and Accu-Chek Guide control solutions*.

*Some items may not be included in the kit. They are a separate purchase.

WARNING



- Choking hazard. Small parts. Keep away from children under the age of 3 years.
- Keep new and used batteries away from children. Ingestion or insertion into the body may cause chemical burns, perforation of soft tissues, and death. Severe burns may occur within 2 hours of swallowing. If you think a battery might have been swallowed or placed inside any part of the body, seek medical attention immediately.
- If the battery compartment does not close securely, stop using the product and keep it away from children. Contact Roche.

- Any object coming into contact with human blood is a potential source of infection (see: Clinical and Laboratory Standards Institute: Protection of Laboratory Workers from Occupationally Acquired Infections; Approved Guideline – Fourth Edition; CLSI document M29-A4, May 2014).

WARNING – KEEP BATTERIES OUT OF REACH OF CHILDREN


In Australia: If you suspect your child has swallowed or inserted a button battery, immediately call the Australian 24-hour Poisons Information Centre on 13 11 26 for fast, expert advice.

Introduction


Why Regular Blood Glucose Testing Is Important

Testing your blood glucose regularly can make a big difference in how you manage your diabetes every day. We have made it as simple as possible.

Important Information About Your New Meter

- The meter may prompt you to choose a language and the time format (12-hour or 24-hour clock) the first time you turn it on.
- Before you begin testing, turn the meter on by briefly pressing  so the meter can synchronise its time and date with the paired pump.
- Sample data screens are shown throughout the manual. Your data will differ.
- If you follow the steps in this manual but still have symptoms that do not seem to match your test results, or if you have questions, talk to your healthcare professional.

The Accu-Chek Guide Link Meter

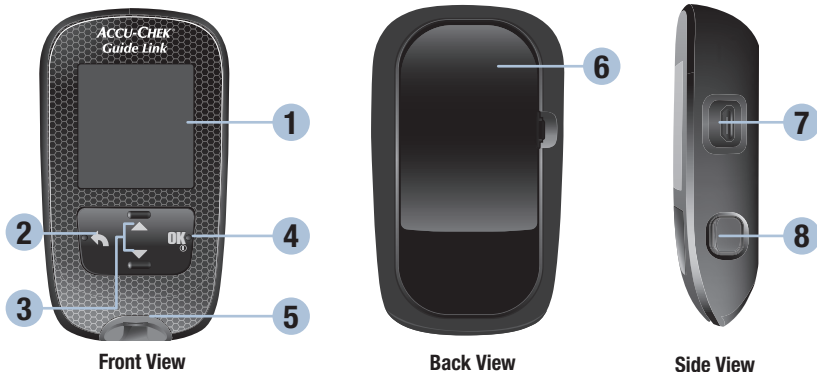

1. Display

Shows results, messages, and test results stored in memory.

2. Back Button

Returns to a previous display or field.

3. Up Arrow and Down Arrow Buttons

Press to move between menu options or to increase or decrease numbers.

4. Power/Set/OK Button

Turns meter on or off and sets options.

5. Test Strip Slot with Light

Insert test strip here.

6. Battery Door

Flip open to replace batteries.

7. Micro USB Port

USB functionality is not available for this meter.

8. Test Strip Ejector

Press to remove test strip.

1

Your New Meter



(for example)



Test Strip



13

9. Test Strip Container*

10. Metallic End

Insert this end into meter.

11. Yellow Edge

Touch blood drop or control solution here.





12. Control Solution Bottle*

13. Batteries

*Some items may not be included in the kit.
They are a separate purchase.


Button Functions

Here are the functions of the back, arrow, and Power/Set/OK buttons on the meter. These functions are used throughout this manual. See the chapter *Meter Settings* for specific instructions on setting up the meter.


Button	Function
 Back Button	Return to the previous display. Return to the previous field.
 Up Arrow and Down Arrow Buttons	Navigate up and down in a menu. Increase or decrease a number.
 Power/Set/OK Button	Press briefly to turn the meter on. Press and hold to turn the meter off. Press to select an option. Press to move to the next field or display. Press to save an option. With the meter off, press and hold to check the meter display. 





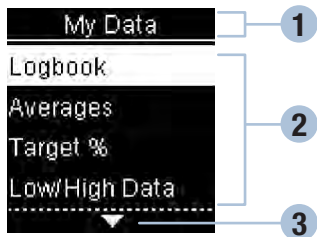


1 Your New Meter

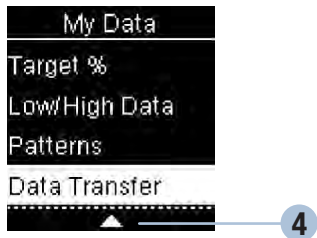

Meter Menus



- 1 Title of display or menu
- 2 Menu options
- 3 Scroll down
- 4 Scroll up



Display	Description
	There are more menu options listed beneath the last option. Press  on the meter to view the options.
	There are more menu options listed above the first option. Press  on the meter to view the options.
	There are more menu options listed above and below the options. Press  or  on the meter to view the options.













Display	Description
	<p>Highlighted option (Test)</p> <p>Press OK to enter the Test menu.</p>
	<p>Highlighted field (Hour)</p> <p>Press ▼ or ▲ to increase or decrease the hour. Press OK to set the hour and move to the minutes field.</p>












1

Your New Meter

Symbols

Here is a list of the symbols on the meter display.


Symbols	Description
	Above target range
	After meal
	Bedtime
	Before meal
	Below target range
	Blood glucose test
	Checkmark / Control test OK / Selected option or setting
	Control bottle
	Control test not OK
	Edit
	Error
	Fasting

Symbols	Description
	Flight mode
	Help
	Low battery
	My data
	No comment
	Other
	Overall
	Settings
	Test reminder
	Warning
	Within target range

1 Your New Meter

Setting the Language and Time Format


The meter may prompt you to choose a language and time format (12-hour or 24-hour clock) the first time you turn it on.

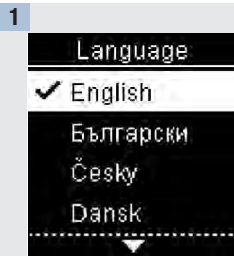

Turn the meter on by briefly pressing **ON**. **Language** appears.



Press **←** or **→** to highlight the language. Press **OK** to set the desired language and continue.

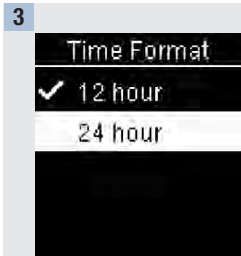

If the meter prompts you to set the time format:

Time Format appears.

Press **←** or **→** to highlight **12 hour** (am/pm) or **24 hour**.


Press **OK** to set the option and continue to **Main Menu**.

NOTE

If you select the wrong language and cannot correct it, contact Roche.

⚠ WARNING

Blood glucose results can be displayed in either mg/dL or mmol/L. The back label of the meter shows the unit of measurement. If the meter shows the wrong unit, contact Roche. If you do not know which unit of measurement is correct for you, contact your healthcare professional. Using the wrong unit of measurement may cause misinterpretation of your actual blood glucose level and may lead to improper therapy.

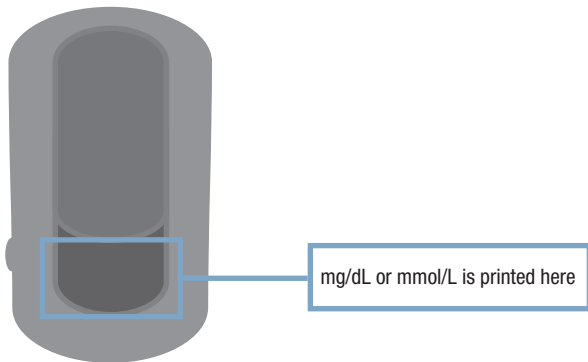

2 Blood Glucose Tests

Using the Accu-Chek Guide Link Meter System

- Use only Accu-Chek Guide test strips.
- Use the test strip immediately after removing it from the test strip container.
- Do not apply blood or control solution to the test strip before inserting it into the meter.
- Close the test strip container tightly immediately after removing a test strip to protect the test strips from humidity.
- Store the unused test strips in their original test strip container with the cap closed.
- Check the use by date on the test strip container. Do not use the test strips after that date.
- Store the test strip container and meter in a cool, dry place such as a bedroom.
- Refer to the test strip package insert for test strip storage and system operating conditions.

WARNING

Do not store test strips in high heat and moisture areas (bathroom or kitchen)! Heat and moisture can damage test strips.

Performing a Blood Glucose Test with Blood from Your Fingertip

NOTE

- Before you perform your first blood glucose test, set up the meter correctly.
- You need the meter, a test strip, and a lancing device with a lancet inserted to perform a blood glucose test.
- There are 2 ways to start a blood glucose test.
 - Insert a test strip into the meter.
 - Turn the meter on by briefly pressing **OK**. Select **Test** > **OK**.

1



Wash your hands with warm soapy water and dry thoroughly.
Prepare the lancing device.

2




Check the use by date on the test strip container.
Do not use test strips past the use by date.

3




Remove a test strip from the test strip container.
Close the cap tightly.

4

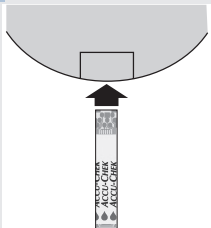

Insert the metallic end of the test strip into the meter.

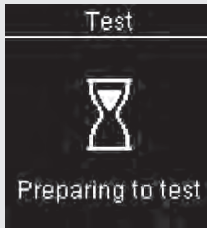

The meter turns on. **Preparing to test** appears.

5



When **Apply drop** appears, prick your finger with the lancing device.



2


Blood Glucose Tests

6



Gently squeeze your finger to assist the blood flow. This helps you get a blood drop.

7

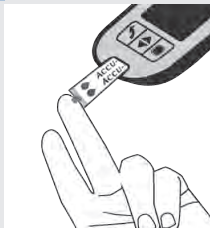

Touch the **yellow edge** of the test strip to the blood drop. Remove your finger from the test strip when **Analyzing** appears. Do not put blood on top of the test strip.



8




or



The test result appears on the display.


You have the option of adding a comment to the test result by pressing  OR proceed to Step 11 to complete the test.

9

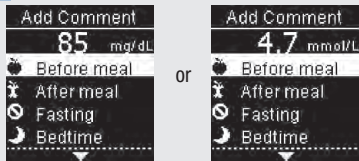

Add Comment appears. Press to highlight a comment. Press to set the comment for the test result. See the Adding Comments to Blood Glucose Results section of this chapter for details.

10

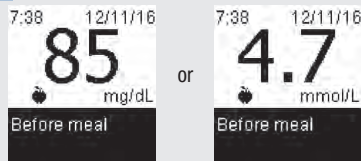

The final result appears with the comment symbol. Press or to set the comment and return to **Main Menu**.

To change the comment, press to select the comment. Press to return to **Add Comment**.

11







Press here to eject test strip

Remove and discard the used test strip by pulling the test strip out of the meter or by pushing the test strip ejector on the side of the meter.

2 Blood Glucose Tests

Blood Glucose Warnings

If your blood glucose result is outside the measuring range of the meter, a warning is displayed. Press **OK** to acknowledge the LO or HI warning, OR the meter automatically moves to the LO or HI result display.

<p>⚠ WARNING LO Result</p> <p>Retest BG. Contact your healthcare professional.</p>	  <p>9:38 12/11/16 LO Add Comment</p>	<p>Blood glucose may be lower than the measuring range of the system. See the Unusual Blood Glucose Results section of this chapter.</p>
<p>⚠ WARNING HI Result</p> <p>Consider checking BG, ketones, and insulin.</p>	  <p>9:38 12/11/16 HI Add Comment</p>	<p>Blood glucose may be higher than the measuring range of the system. See the Unusual Blood Glucose Results section of this chapter.</p>

Adding Comments to Blood Glucose Results

NOTE

Analysing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.







Overview

It is very important to have the correct time and date set in the meter. Having the correct time and date setting helps ensure accurate interpretation of information by you and your healthcare professional.


- You may add comments to blood glucose results to help you and your healthcare professional analyse patterns detected by the meter (see the Patterns section in the chapter Meter Settings for details).
- If Patterns is set to **On**, once a pattern is detected you may NOT change the comment attached to a blood glucose result (see the Patterns section in the chapter Meter Settings for details).
- Adding a comment saves the comment and the symbol with the blood glucose result.

2 | Blood Glucose Tests


Here is a list of comment symbols that can be added to a blood glucose result.

Symbols	Name	Description
	Before meal	If Patterns is On , select Before breakfast, lunch, dinner, or snack (see the following page for adding comments with Patterns On).
	After meal	If Patterns is On , select After breakfast, lunch, dinner, or snack (see the following page for adding comments with Patterns On).
	Fasting	Select Fasting for no caloric intake for at least 8 hours.*
	Bedtime	
	Other comment	You can use this comment to mark an event such as an Alternative Site Testing (AST) result or exercise.
	No entry	1. You do not want to add a comment. 2. You want to remove a comment for the current blood glucose result.

*American Diabetes Association: Standards of Medical Care in Diabetes-2016.

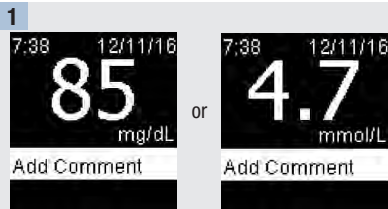

After performing a blood glucose test, the test result is displayed on the screen with **Add Comment** highlighted. Press **OK** to add a comment.

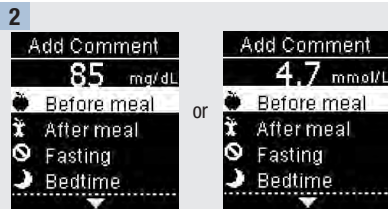

The **Add Comment** menu appears. Press **▼** to select the desired comment (the example here is **Before meal**). Press **OK** to set the comment.





If Patterns is On:



If **Patterns** is on and you select **Before meal** or **After meal**, press **▼** to select a specific meal (**Breakfast**, **Lunch**, **Dinner**, or **Snack**). Press **OK** to set the selected meal for the test result.

2 Blood Glucose Tests

3



The final result appears with the comment symbol. When automatically sending results to a paired pump, press  or  to set the comment, send the result, and return to **Main Menu**.

To change the comment, press  to select the comment. Press  to return to **Add Comment**.

WARNING

When using a test result for calibrating a continuous glucose monitoring system or to make insulin dosing calculations, confirm that the result displayed on the pump matches the result displayed on the meter.

Performing a Blood Glucose Test with Blood from Your Palm, Forearm, or Upper Arm (Alternative Site Testing)

You have the option of obtaining a blood sample from other sites on your body besides the fingertip. Alternative sites include the palm, forearm, and upper arm.

Blood obtained from the fingertip and palm can be used at any time to perform a blood glucose test.

If blood from the forearm or upper arm is used, there are certain times when testing is not appropriate. This is because your blood glucose level changes faster in your fingertip and palm than in the forearm and upper arm. These differences may cause you to misinterpret your actual blood glucose level, leading to improper therapy and potential adverse health effects.

Read the next section before you try testing from the forearm or upper arm.

You may perform a forearm or upper arm test	<ul style="list-style-type: none">• immediately before a meal.• while fasting.
You may NOT perform a forearm or upper arm test	<ul style="list-style-type: none">• up to 2 hours following a meal, when blood glucose values can rise quickly.• after injecting bolus insulin, when blood glucose values can decrease rapidly.• after exercise.• if you are sick.• if you think your blood glucose is low (hypoglycaemia).• if you sometimes do not notice when your blood glucose is low.

WARNING

Do not use alternative site testing (AST) to calibrate a continuous glucose monitoring system or to make insulin dosing calculations.

If you are interested in AST, talk to your healthcare professional first.

To obtain an AST cap and detailed AST instructions, contact Roche.

2 | Blood Glucose Tests

Unusual Blood Glucose Results

If your blood glucose result does not match how you feel, check this list to help solve the problem.

Troubleshooting Checks	Action
Did you wash your hands?	Wash your hands with warm soapy water and dry thoroughly. Repeat the blood glucose test with a new test strip.
Were the test strips expired?	Discard the test strips if they are past the use by date. Repeat the blood glucose test with an unexpired test strip.
Was the cap on the test strip container always closed tightly?	Replace the test strips if you think the test strip container was uncapped for some time. Repeat the blood glucose test.
Was the test strip used immediately after it was removed from the test strip container?	Repeat the blood glucose test with a new test strip.
Were the test strips stored in a cool, dry place?	Repeat the blood glucose test with a properly stored test strip.
Did you follow the directions?	See the chapter Blood Glucose Tests and repeat the blood glucose test. Contact Roche if you still have problems.
Are the meter and test strips working properly?	Perform a control test. See the chapter Control Tests for instructions.
Are you still unsure of the problem?	Contact Roche.

Symptoms of Low or High Blood Glucose

Being aware of the symptoms of low or high blood glucose can help you understand your test results and decide what to do if they seem unusual.

Low blood glucose (hypoglycaemia): Symptoms of hypoglycaemia may include, but are not limited to, anxiety, shakiness, sweating, headache, increased hunger, dizziness, pale skin colour, sudden change in mood or irritability, fatigue, difficulty concentrating, clumsiness, palpitations, and/or confusion.

High blood glucose (hyperglycaemia): Symptoms of hyperglycaemia may include, but are not limited to, increased thirst, frequent urination, blurred vision, drowsiness, and/or unexplained weight loss.

WARNING

If you are experiencing any of these symptoms, or other unusual symptoms, test your blood glucose from the fingertip or palm. If your blood glucose result is displayed as LO or HI, contact your healthcare professional immediately.

2

Blood Glucose Tests

When to Perform a Control Test

Performing a control test lets you know the meter and test strips are working properly. You should perform a control test when:

- you open a new test strip box.
- you left the test strip container open.
- you think the test strips are damaged.
- you want to check the meter and test strips.
- the test strips were stored in extreme temperatures, humidity, or both.
- you dropped the meter.
- your test result does not match how you feel.
- you want to check if you are performing the test correctly.

About the Control Solutions

- Use only Accu-Chek Guide control solutions.
- Close the control solution bottle tightly after use.
- Write the date you open the control solution bottle on the bottle label. The control solution must be discarded 3 months from the date the control solution bottle was opened (discard date) or on the use by date on the bottle label, whichever comes first.
- Do not use control solution that is past the use by or discard date.
- Refer to the control solution package insert for control solution storage conditions.
- The meter automatically recognises the difference between the control solution and blood.
- The control results are not displayed in memory.
- The control solution can stain fabric. Remove stains by washing with soap and water.

3 Control Tests

Performing a Control Test

You need the meter, a test strip, and control solution Level 1 or Level 2.

1




Check the use by date on the test strip container. Do not use test strips past the use by date.

2




Remove a test strip from the test strip container. Close the cap tightly.

3

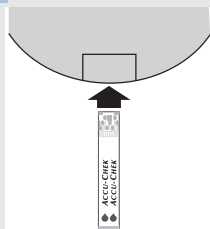

Insert the metallic end of the test strip into the meter. Place the meter on a flat surface.

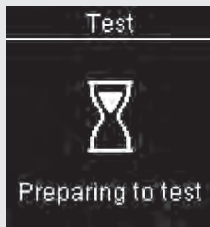

The meter turns on. **Preparing to test** appears.



Apply drop appears.

4




Select the control solution to test. You will enter the level later in the test.

5



Remove the bottle cap. Wipe the tip of the bottle with a tissue. Squeeze the bottle until a tiny drop forms at the tip.

6

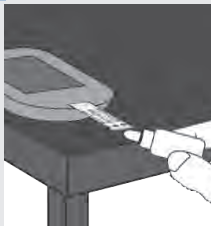

Touch the drop to the **yellow edge** of the test strip. Do not put control solution on top of the test strip.



Analyzing appears when there is enough control solution in the test strip.

3 Control Tests

7



Control Result and the control bottle symbol appear. Press or to select the control level you tested. If you do not select a level, the control result is saved without a control level. Press .

8



Within range and appear if the control result is within range.

Out of range and appear if the control result is out of range.

9



Wipe the tip of the bottle with a tissue. Cap the bottle tightly.

Remove and discard the used test strip.

NOTE

Control results are not transmitted to the pump.

- Do not calibrate your continuous glucose monitoring device from a control result.
- Do not calculate a bolus based on a control result.

The meter turns off 90 seconds after a successful test or 15 seconds after the test strip is removed, provided no other action is taken.

3 Control Tests

Understanding Out-of-Range Control Results

WARNING

The control ranges are printed on the test strip container label. If the control result is out of range, check this list to help solve the problem.

Troubleshooting Checks	Action
Were the test strips or control solutions expired?	Discard the test strips or control solution if either is past the use by date. If the control solution was opened more than 3 months ago, discard it. Repeat the control test with an unexpired test strip and an unexpired control solution.
Did you wipe the tip of the control solution bottle before use?	Wipe the tip of the bottle with a tissue. Repeat the control test with a new test strip and a fresh drop of control solution.
Were the caps on the test strip container and the control solution bottle always closed tightly?	Replace the test strips or control solution if you think either was uncapped for some time. Repeat the control test.
Was the test strip used immediately after it was removed from the test strip container?	Repeat the control test with a new test strip and a fresh drop of control solution.
Were the test strips and control solutions stored in a cool, dry place?	Repeat the control test with a properly stored test strip or control solution.
Did you follow the directions?	Read the chapter Control Tests and repeat the control test.
Did you choose the correct control solution level, either 1 or 2, when you performed the control test?	If you chose the wrong control solution level, you can still compare the control result to the range printed on the test strip container.
Are you still unsure of the problem?	Contact Roche.

Overview

You can adjust the following settings in the meter for your personal preferences. Refer to the sections later in this chapter for details and how to set the options.

Setting	Options	Function
Time/Date	Time / Date	Set the time and date. NOTE Feature is only available if the meter is not paired with a pump. If paired with a pump, the meter time and date are automatically set to the pump's time and date during communication.
Beeper	On / Off	Select On or Off.
Wireless	Select wireless communication settings. See the chapter Wireless Communication and Meter Pairing.	

4

Meter Settings

Setting	Options	Function
Target Ranges	Off / Single Range / 2 Ranges	Select the blood glucose target range appropriate for you. NOTE Consult your healthcare professional for the appropriate target range for you. Off – no target range arrow symbols appear with blood glucose result. Patterns is Off (see the Patterns section of this chapter for details).
	70–160 mg/dL (pre-set target range) 3.9–8.9 mmol/L (pre-set target range)	Single Range – blood glucose results are marked as above, within, or below range based on the single target range set in the meter.
	Before Meal Range 70–110 mg/dL (pre-set target range) 3.9–6.1 mmol/L (pre-set target range) After Meal Range 70–160 mg/dL (pre-set target range) 3.9–8.9 mmol/L (pre-set target range)	2 Ranges – set Before Meal and After Meal ranges. Blood glucose results are marked as above, within, or below range based on the 2 target ranges (Before Meal and After Meal) set in the meter.
Patterns	On / Off	On – a pattern is detected when 2 below-target or 3 above-target test results with the same comment are detected within a 7-day period.
Reminders	On / Off / Edit time	On – set up to 4 reminders per day to remind you to test.

Setting	Options	Function
Post Meal	On / Off / Edit time	On – reminds you to perform an after meal blood glucose test.
Last Result	On / Off	Select whether the previous blood glucose result (within the past 24 hours) appears with the current blood glucose result. On – the previous blood glucose result appears with the current blood glucose result. Off – only the current blood glucose result appears.
Language		Select the language for the meter.
Time Format	12 hour / 24 hour	Select the clock format for the meter.

4 Meter Settings

Time/Date

Time/Date

NOTE


Feature is only available if the meter is not paired with a pump. If paired with a pump, the meter time and date are automatically set to the pump's time and date during communication.

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2

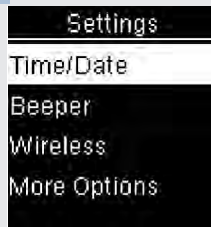

Time/Date is highlighted. Press **OK**.

3



Press **▼** or **▲** to adjust each field. Press **OK** to set and move to the next field. Set **am** or **pm** if necessary.

Press **OK** to save and return to the previous menu.

Beeper

The beeper prompts you:

- when a test strip is inserted.
- to apply blood or control solution to the test strip.
- when enough blood or control solution is drawn into the test strip.
- when the blood glucose or control test is complete.
- when the meter is turned on.
- when a button is pressed.
- when it is time to perform a test (if Reminders or Post Meal reminders are **On**).
- when the batteries are inserted.
- when there are no stored blood glucose results or there is an invalid record in the logbook.
- when there are no errors in the error log.
- if an error occurred (even if the beeper is off, it still beeps for an error).

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2



Press **▼** to highlight **Beeper**. Press **OK**.

3

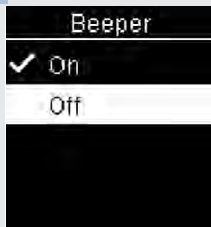

Press **▼** or **▲** to highlight **On** or **Off**. Press **OK** to move **✓** to the option.

Press **OK** to set the option and return to the previous menu.

4

Meter Settings

Target Ranges




Target Ranges

Your healthcare professional can tell you what blood glucose range is appropriate for you. It is very important to stay within your target range.

Target Ranges can be set from a lower limit of 60–100 mg/dL (3.3–5.5 mmol/L) to an upper limit of 101–300 mg/dL (5.6–16.6 mmol/L).

Options	Function
Off	Arrow symbols for above, within, or below target ranges do not appear with the blood glucose results.
Single Range	Set lower limit and upper limit for the Target Range. You will be prompted to turn on Patterns if desired (see the Patterns section of this chapter for details).
2 Ranges	Set lower limits and upper limits for Before Meal and After Meal target ranges. You must mark your blood glucose results with a comment for the meter to detect above, within, or below Before Meal or After Meal test results (see the Adding Comments to Blood Glucose Results section in the chapter Blood Glucose Tests for details). You will be prompted to turn on Patterns if desired (see the Patterns section of this chapter for details).

When Target Ranges is on, the following symbols appear with blood glucose results.

Symbol	Meaning
	The blood glucose result is below the target range.
	The blood glucose result is within the target range.
	The blood glucose result is above the target range.

 **WARNING**

This function is no substitute for hypoglycaemia training by your healthcare professional.

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2



Press **▼** to highlight **More Options**. Press **OK**.

3



Target Ranges is highlighted. Press **OK**.



4 Meter Settings

Target Ranges

4



Press **▼** or **▲** to highlight **Off**, **Single Range**, or **2 Ranges**. Press **OK** to select an option (the example here is **2 Ranges**).


5



Press **▼** or **▲** to adjust the lower limit of the **Before Meal** target range. Press **OK** to set and move to the next field.


Continue to set the upper limit for the **Before Meal** range and the lower and upper limits for the **After Meal** target range. Press **OK**.

6

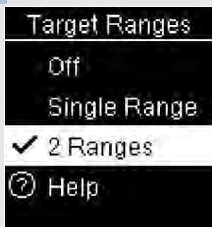

The **✓** appears next to the option you selected.

Press **Back** to return to the previous menu.

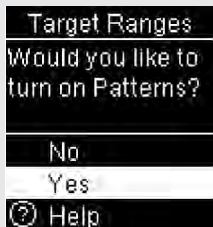

The meter may prompt you to turn on Patterns.

Press **▼** or **▲** to highlight **Yes** or **No**.

Press **OK** to set the option and return to the previous menu (see the **Patterns** section of this chapter for details).

NOTE

Analysing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.

Patterns

A Pattern is detected by the meter when 2 below-target (Low Pattern) or 3 above-target (High Pattern) test results with the same comment are detected within a 7-day period.

- The meter does NOT detect a pattern for the “other” comment added to blood glucose results.
- It is very important to have the correct time and date set in the meter. Having the correct time and date setting helps ensure accurate interpretation of information by you and your healthcare professional.
- Only blood glucose results marked with comments will be included in Patterns. If LO or HI test results are marked with comments, the results become part of Patterns (see the chapter Blood Glucose Tests for more details).
- Target Ranges must be set in the meter to use Patterns. If Target Ranges is not set, the meter prompts you to set them.

4 Meter Settings

Patterns

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2




Press **▼** to highlight **More Options**. Press **OK**.

3




Press **▼** to highlight **Patterns**. Press **OK**.

4

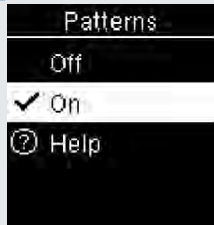


Press **▼** to highlight **On**. Press **OK** to move **✓** to the option. Press **OK**.

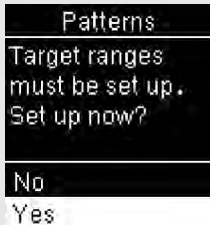

This message appears if Target Ranges is Off:

Press **▼** to highlight **Yes**. Press **OK**.

(To turn Patterns **Off**, select **No**. Press **OK** to return to **Patterns**.)

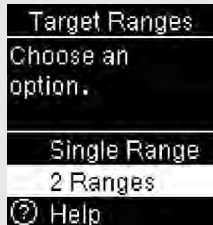


Target Ranges appears on the display (see the Target Ranges section of this chapter for details on setting target ranges).

NOTE


When automatically sending results to a paired pump, select the **View later** option to send the test result without delay.

If Patterns is **On** and a new pattern is detected with a blood glucose result, a message appears on the display.

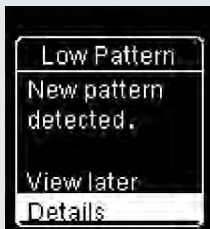

Press **▼** to select **Details** to view the blood glucose results that make up that pattern.


or

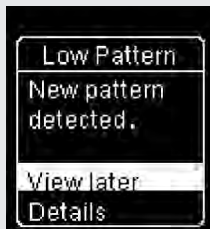

Press **⏏** to highlight **View later**.

Press **⏏** to return to the previous screen.

Reminders

You can set up to 4 general test Reminders per day to remind you to test. A series of beeps sound and  is displayed for Reminders set in the meter.

Reminders:

- turn off by inserting a test strip or pressing any button.
- are postponed until the next test reminder if a test was performed within 15 minutes of a test reminder.
- do not appear/beep if the meter is on at the test reminder time.
- do not appear/beep if the meter is connected and communicating to the paired pump.
- do not beep if the meter's beeper is set to off.
- do not appear/beep if the batteries need to be replaced.

Set Reminders

- Reminder times are pre-set in the meter for 8:00, 12:00, 18:00, and 22:00. You may change reminder times per the instructions on the following pages.
- If a general test Reminder is set for the same time as a Post Meal reminder, the Post Meal reminder will appear/beep instead of the general Reminder (see the Post Meal Reminders section of this chapter for details).

4

Meter Settings


Reminders

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2

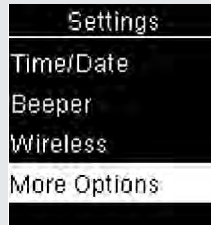


Press **▼** to highlight **More Options**. Press **OK**.

3




Press **▼** or **▲** to highlight **Reminders**. Press **OK**.

4

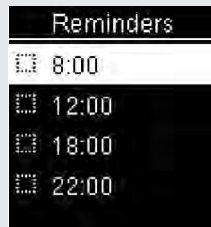


Pre-set reminder times appear on the display. Press **▼** to highlight a Reminder time. Press **OK**.

5

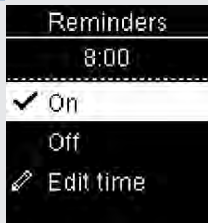

The pre-set reminder time appears. Press **▼** or **▲** to select **On** or **Off**. Press **OK** to move **✓** to the option.

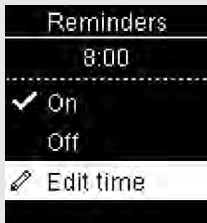


To change the Reminder time shown:

Press **▼** to highlight **Edit time**. Press **OK**.



Press **▼** or **▲** to adjust each field. Press **OK** to move to the next field. Press **OK** to return to the previous menu.

6

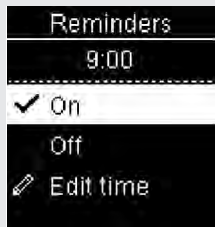


A **✓** indicates the Reminder is set for the time shown and automatically set to **On**.

To turn the reminder off, press **▼** to highlight **Off**. Press **OK**.

Press **OK** to return to **Reminders**.


7

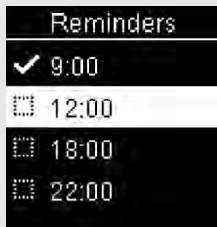

The reminder time appears.

Press **▼** to continue to set additional reminders or press **OK** to return to **More Options**.

Post Meal Reminders

Post Meal reminders can be set to remind you to test again later when you add a Before Meal comment to a blood glucose result. When a reminder occurs, a series of beeps sound and  is displayed.


Post Meal reminders:

- turn off by inserting a test strip or pressing any button.
- are postponed until the next reminder if a test was performed within 15 minutes of a test reminder.
- do not appear/beep if the meter is on at the test reminder time.
- do not appear/beep if the meter is connected and communicating to the paired pump.
- do not beep if the meter's beeper is set to off.
- do not appear/beep if the batteries need to be replaced.

Set Post Meal Reminders

- Adding a Before meal comment to a blood glucose result sets a Post Meal reminder in the meter.
- Marking blood glucose results with an After meal comment provides more information about your test results to help you and your healthcare professional in the management of your diabetes.
- Talk to your healthcare professional to determine your Post Meal test time.
- Select 1 hour, 1.5 hours, or 2 hours for Post Meal reminders to occur.

1

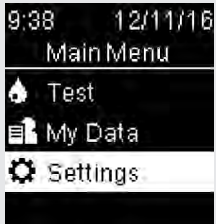

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2



Press **▼** to highlight **More Options**. Press **OK**.

3



Press **▼** to highlight **Post Meal**. Press **OK**.

4

Meter Settings

Post Meal Reminders

4



Press **▼** or **▶** to highlight **On**.
 Press **↵** to move ✓ to the option.

5



Press **▼** or **▶** to highlight **Edit time**.
 Press **↵**.

6




Press **▼** or **▶** to highlight **1 hour**, **1.5 hours**, or **2 hours** after a meal for **Post Meal** reminders to occur.

Press **↵** to move ✓ to the option.

Press **⏪** to set and return to the previous menu.


Last Result

Select whether the previous blood glucose result appears with the current blood glucose result. **Test results older than 24 hours do not appear.**

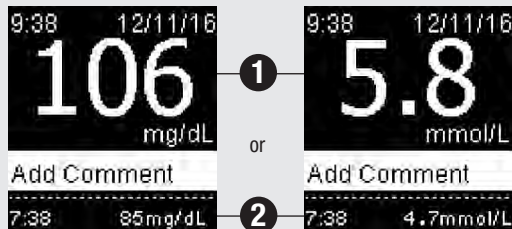

Off – only the current blood glucose result ❶ appears.

On – the previous blood glucose result ❷ appears with the current blood glucose result.

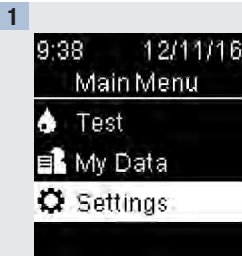

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

4 Meter Settings

Last Result

2




Press **▼** to highlight **More Options**. Press **OK**.

3



Press **▼** to highlight **Last Result**. Press **OK**.

4

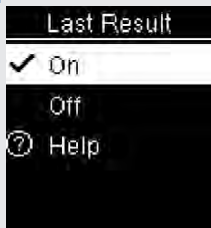

Press **▼** or **▲** to highlight **On** or **Off**. Press **OK** to move **✓** to the option.

Press **OK** to set the option and return to the previous menu.

Language

Choose the language that appears on the meter.

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2




Press **▼** to highlight **More Options**. Press **OK**.

3



Press **▼** to highlight **Language**. Press **OK**.

4

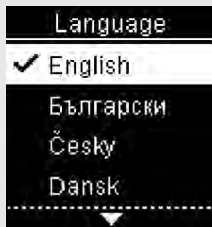

Press **▼** or **▲** to highlight the desired language. Press **OK** to move **✓** to the option.

Press **OK** to set the language and return to the previous menu.

4 Meter Settings

Time Format

Time Format

Choose the time format (12-hour or 24-hour clock) that appears on the meter.

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2



Press **▼** to highlight **More Options**. Press **OK**.

3



Press **▼** to highlight **Time Format**. Press **OK**.

4



Press **▼** or **▲** to highlight **12 hour** (am/pm) or **24 hour**. Press **OK** to move **✓** to the option.

Press **OK** to set the time format and return to the previous menu.

Overview

- Blood glucose results are stored from the newest to the oldest.
- The meter automatically stores up to 720 blood glucose results in memory with the time and date of the test and any test result comments.
- Once 720 blood glucose results are in memory, adding a new blood glucose result deletes the oldest blood glucose result.
- Only test results that have been marked with a fasting, before meal, after meal, or bedtime comment are included in the average for that comment.
- All test results are included in the overall 7, 14, 30, and 90-day averages regardless of what comment is added.
- Control results are not included in the averages or blood glucose reports.

WARNING

Do not change your therapy based on an individual test result in memory. Talk to your healthcare professional before changing therapy based on test results in memory.

5 Review Your Data

Logbook


Logbook

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

2

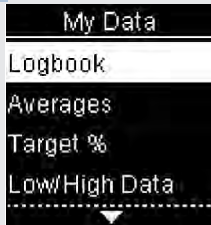

Logbook is highlighted. Press **OK**.

3

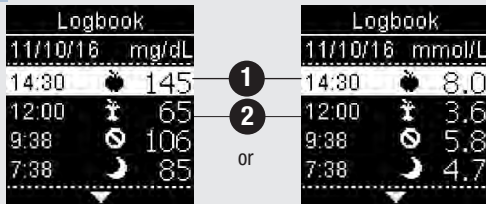


Press **▼** or **▶** to scroll through **Logbook**.

The most recent test result **1**.

The 2nd most recent test result **2**.


- 4 To view details about a test result, press  or  to highlight the test result. Press . Test result details shown below only appear if **Target Ranges** is on or comments were added to a test result.

1

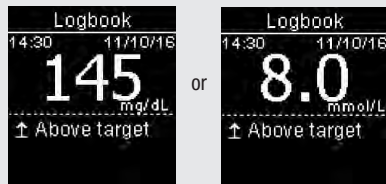

The most recent test result.

2

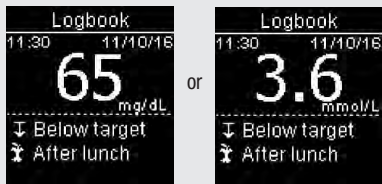


The 2nd most recent test result.

5


Review Your Data

Averages

Averages

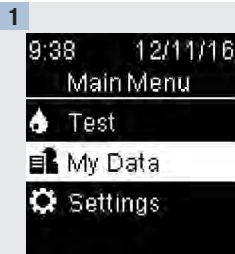


Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

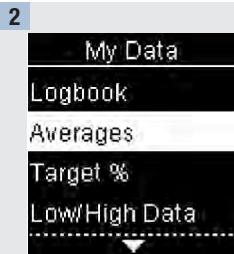


Press **▼** to highlight **Averages**. Press **OK**.



Press **▼** to highlight a category (the example here is **Overall**). Press **OK**.

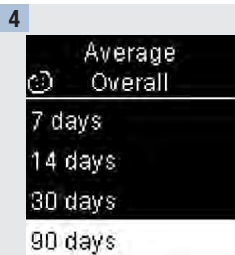

Press **▼** to highlight a time period (the example here is **90 days**). Press **OK**.

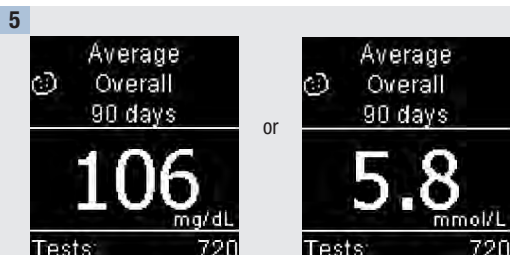


Press **↶** to return to the previous menu if you want to review a different time period OR press **▼** or **▶** to move through different averages.

Target Percent (%)

Target Percent (%) allows you to view the percentage of your Overall, Before meal, After meal, Fasting, and Bedtime blood glucose results that are above, within, or below your target ranges.

- Target % results can be viewed for 7, 14, 30, or 90-day time periods.
- Target Ranges must be set in the meter to review Target % results.


1



9:38 12/11/16
Main Menu
Test
My Data
Settings

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.


2



My Data
Logbook
Averages
Target %
Low/High Data

Press **▼** to highlight **Target %**. Press **OK**.

3



Target %
Overall
Before meal
After meal
Fasting

Press **▼** to highlight a category (the example here is **Before meal**). Press **OK** to select the option.

5

Review Your Data

Target Percent (%)

4


Target %
🍷 Before meal
7 days
14 days
30 days
90 days

Press  to highlight a time period (the example here is **90 days**). Press .

5

Target %	
🍷 Before meal	
90 days	
↑ Above	15%
↕ Within	60%
↓ Below	25%
Tests:	750

The **Target %** appears (for the **Before meal** example). The number of total tests included in the Target % appears at the bottom of the display.

Press  to return to the previous menu.

NOTE

Analysing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.

Low/High Data

Your healthcare professional can tell you what blood glucose range is appropriate for you. It is very important to stay within your target range.

- Target Ranges must be set in the meter to track Low/High Data test results (see the Target Ranges section in the chapter Meter Settings for details).
- Low BG or High BG Data includes only results that fall above or below the target ranges set in the meter.
- Low BG or High BG Data is tracked in the meter for 30 days.








5

Review Your Data

Low/High Data

You can select Low BG and High BG results for Overall, Before meal, After meal, Fasting, or Bedtime blood glucose results.

Symbol	Name	Description
	Overall	Includes low and high blood glucose results based on Target Ranges set in the meter.
	Before meal	You may view low or high test results marked with a Before meal comment for Overall, Before breakfast, Before lunch, Before dinner, and Before snack blood glucose results.*
	After meal	You may view low or high test results marked with an After meal comment for Overall, After breakfast, After lunch, After dinner, and After snack blood glucose results.*
	Fasting	Includes low or high fasting blood glucose results marked as Fasting in comments.
	Bedtime	Includes low or high bedtime blood glucose results marked as Bedtime in comments.

*Test results for Before and After specific meals are only available if Patterns is set to **On**.

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

2



Press **▼** to highlight **Low/High Data**. Press **OK**.


3

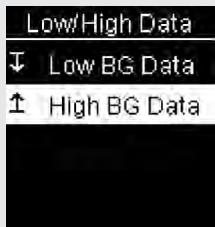

If Target Ranges is on:

Press **▼** to select **Low** or **High BG Data** (the example here is **High BG Data**). Press **OK**.

or

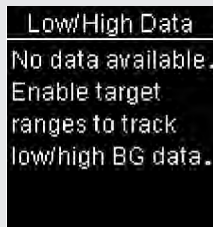

If Target Ranges has NOT been on within the last 30 days:

This message appears on the meter (to turn on Target Ranges see the Target Ranges section in the chapter Meter Settings for details).

5

Review Your Data

Low/High Data

4



Press **▼** to highlight a category (the example here is **Before meal**). Press **OK**.




If test results with detailed meal comments are saved in the Logbook:

The meter may prompt you to select detailed categories to view. Press **▼** to highlight a category (the example here is before **Breakfast**). Press **OK**.

5



or

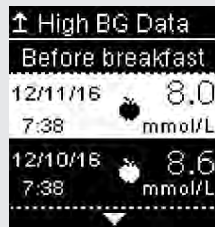

The selected data appears (the example here is **High BG Data**). Press **▼** to scroll through the test results.

Press **◀** to return to the previous menu.

NOTE

Analysing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.

Patterns

- Patterns displays only active Low Patterns or High Patterns based on comments added to blood glucose results within the last 7 days.
- A Pattern is generated when 2 below-target or 3 above-target test results with the same comment are detected within a 7-day period.

Patterns may be viewed on the meter in 3 ways:




when a Patterns option is displayed at the bottom of **Main Menu**.

or



from **My Data** on Main Menu.

or

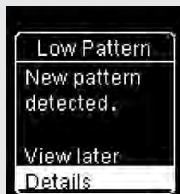

if a **New pattern detected** message appears on the display when performing a blood glucose test.

5

Review Your Data

Patterns

Patterns detected by the meter may be displayed on the Main Menu as:


Patterns	High and low patterns have been detected	Patterns may include the following blood glucose results marked with comments: Before breakfast, After breakfast, Before lunch, After lunch, Before dinner, After dinner, Before snack, After snack, Fasting, or Bedtime (see the Adding Comments to Blood Glucose Results section in the chapter Blood Glucose Tests).
High Pattern(s)	One or more high patterns have been detected	
Low Pattern(s)	One or more low patterns have been detected	
No Patterns	No active pattern based on test results from last 7 days	
Blank	Patterns feature is set to Off	

1



Turn the meter on by briefly pressing **ON**. From **Main Menu**, press **▼** to highlight the **Patterns** option at the bottom of the screen (the example here is **Patterns**). Press **ON**.

2

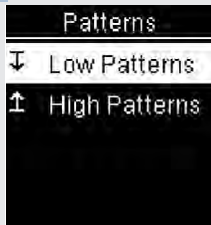

Press **▶** or **◀** to highlight **Low Patterns** or **High Patterns** (the example here is **Low Patterns**). Press **ON** to select the option.

3




Press **▼** to highlight a category to review (the example here is **Before breakfast**). Press **ON**.

4



Press **▼** to scroll through test results.

or

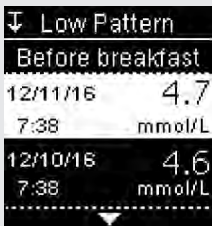

Press **◀** to return to the previous menu.

5

Review Your Data



Overview

You can wirelessly and automatically synchronise your diabetes information with a compatible MiniMed™ Pump with Bluetooth® wireless technology. The process of creating a connection between the meter and the pump is called pairing.

Setting	Options	Function
Flight Mode	On / Off	Select whether wireless communication is available. On – wireless communication is not available. Off – wireless communication is available.
Auto-Send	On / Off	Select whether data is automatically sent to the paired pump after each test. On – data is automatically sent to the pump. Off – data is not automatically sent to the pump.
Pairing	Pair To Pump / Delete Pairing	Select whether to pair a pump or to delete the paired pump.

6

Wireless Communication and Meter Pairing

Pairing


Pairing

The meter can automatically send blood glucose results to a compatible MiniMed™ Pump with Bluetooth® wireless technology. This feature eliminates the need to manually enter your blood glucose result on the pump. Before the meter can send blood glucose results to a pump, the meter and pump must be paired. Refer to the MiniMed™ System User Guide for pairing instructions.

Once the meter and pump are paired, the pairing settings are stored in both devices so that you do not have to repeat the pairing. If communication between the meter and pump is stopped or interrupted for any reason, they will automatically resume communication when both devices are in communication range.

The meter can only be paired with a compatible MiniMed™ Pump. The meter can only be paired with 1 pump at a time. It is not necessary to delete the existing pairing when pairing with a new pump. Any blood glucose results stored on the meter prior to pairing cannot be transferred from the meter to the pump.

Flight Mode

Select whether wireless communication is available or not. When **Flight Mode** is on,  appears in the title bar and wireless communication is not available.

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2




Press **▼** to highlight **Wireless**. Press **OK**.


3

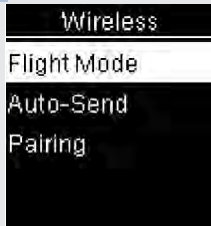

Flight Mode is highlighted. Press **OK**.

4

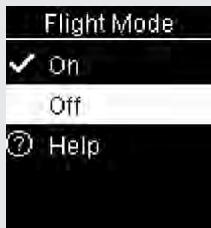

Press **▼** or **▲** to highlight **On** or **Off**. Press **OK** to move  to the option.

Press **OK** to set the option and return to the previous menu.



Main Menu with meter in Flight Mode.

6

Wireless Communication and Meter Pairing

Auto-Send

Auto-Send

Select whether blood glucose results are automatically sent to the paired pump after each test. Test results sent using Auto-Send can be used by the pump system to calibrate a continuous glucose monitoring system or make insulin dosing calculations.

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2




Press **▼** to highlight **Wireless**. Press **OK**.

3

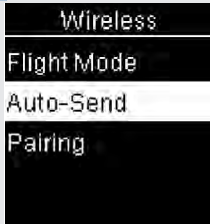

Press to highlight **Auto-Send**. Press .

4

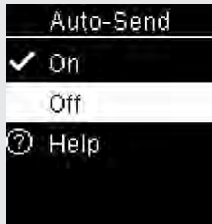

Press or to highlight **On** or **Off**. Press to move to the option.

Press to set the option and return to the previous menu.

 WARNING

Do not use alternative site testing (AST) to calibrate a continuous glucose monitoring system or to make insulin dosing calculations.

6

Wireless Communication and Meter Pairing

Data Transfer

Data Transfer

This feature allows you to transfer data wirelessly from your meter to a paired pump. This feature can be used after the meter and paired pump have not been communicating for a period of time, such as when Flight Mode has been enabled or Auto-Send has been disabled. In this case, there may be blood glucose results that have not been sent to the pump.

Test results sent using Data Transfer cannot be used by the pump system to calibrate a continuous glucose monitoring system or to make insulin dosing calculations.

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

2



Press **▼** to highlight **Data Transfer**. Press **OK**.

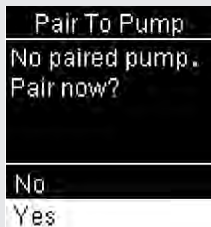

If a pump is not paired with your meter:

The meter prompts you to pair a pump to the meter (see the Pairing section in the chapter Wireless Communication and Meter Pairing for details).

3

Data Transfer



Connecting to
pump 1234567B

Data Transfer



Transferring to
pump 1234567B

Data Transfer



Transferred to
pump 1234567B

The meter transfers the data to the pump.


6 Wireless Communication and Meter Pairing

Delete Pairing


Delete Pairing

This procedure is to delete the pairing on the meter, which ends communication between the meter and the pump. For instructions on deleting the pairing information on the pump, refer to the MiniMed™ System User Guide.


The meter can only be paired with 1 pump at a time. It is not necessary to delete the existing pairing when pairing with a new pump.

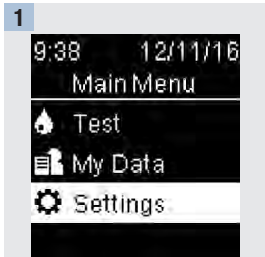

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

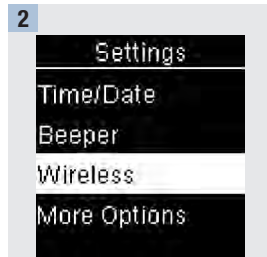


Press **▼** to highlight **Wireless**. Press **OK**.

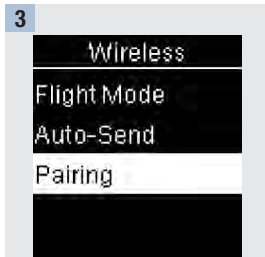

Press **▼** to highlight **Pairing**. Press **OK**.



4

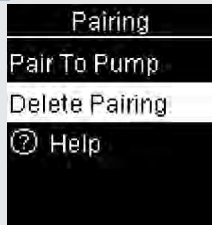

Press **▼** or **▶** to highlight **Delete Pairing**. Press **OK**.

5



Press **OK** to confirm.



The meter shows the deleted pairing.

Meter Maintenance


The meter automatically tests its own systems every time you turn it on and lets you know if something is wrong. See the Error Messages section of this chapter.

If you drop the meter or think the results are not accurate, contact Roche.


WARNING


Keep new and used batteries away from children. See the warning in the Introduction of this User's Manual for additional information.


Changing the Batteries

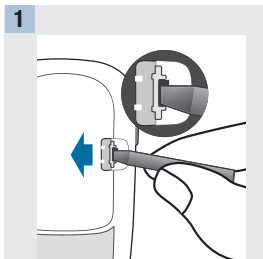


Open the battery door by inserting a narrow object into the slot, pushing the tab in the direction of the arrow, and lifting the door up.

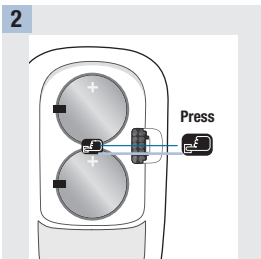

Release the old batteries by pressing the  button. Remove the old batteries. Press and hold the Power/Set/OK button on the front of the meter for at least 2 seconds.

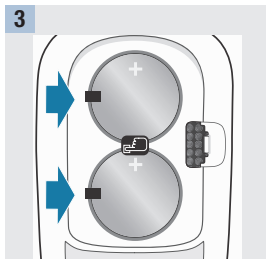

Slide the new batteries under the black tabs and  button, with the (+) side facing up. Put the battery door back in place and snap it closed. Immediately discard the old batteries.

NOTE

- The meter uses two 3-volt lithium batteries, coin cell type CR2032. This type of battery can be found in many stores. It is a good idea to have spare batteries available.
- Always replace both batteries at the same time and with the same brand.
- The logbook data is saved when you replace the batteries.

7 Meter Maintenance and Troubleshooting

Cleaning the Meter

Keep the meter free of dust. If you need to clean or disinfect it, follow these guidelines carefully to help you get the best performance possible.

WARNING

- Do not allow liquid to enter any openings in the meter.
- Do not spray a cleaning solution directly onto the meter.
- Do not immerse the meter in liquid.

1

Make sure the meter is turned off.

2

Gently wipe the meter's surface with a soft cloth slightly dampened (wring out any excess liquid) with one of these cleaning solutions:

- 70 % isopropyl alcohol
- Mild dishwashing liquid mixed with water
- 10 % household bleach solution (1 part bleach plus 9 parts water) made the same day

Error Messages

WARNING

- Never make therapy decisions based on an error message.
- If you have any concerns or see any other error message, contact Roche.



The meter will not turn on or the display is blank.

- Batteries are dead.

Insert new batteries.

- Display is damaged. / Meter is defective.

Contact Roche.


- Extreme temperatures.

Move the meter to a location with a more moderate temperature.




The meter is connected to a USB cable and a test cannot be performed.

Remove the USB cable and perform a test.

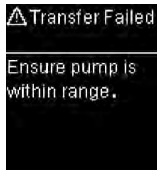

Blood glucose results were not transferred to the paired pump.

Make sure the paired pump is within range of the meter.

 Transfer
Not Allowed

Wireless
communication
not allowed in
flight mode.

Data cannot be sent to the paired pump because the meter is in Flight Mode.


Retry the data transfer when the meter is not in Flight Mode.

 No USB
Features

Remove USB
cable.


USB functionality is not available for this meter.

Remove the USB cable.

 Results
Excluded


One or more
results may be
excluded from
averages.

One or more blood glucose results are excluded from the selected averages because the results are invalid or outside the system measuring range.

 Results
Excluded

One or more
results may be
excluded from
target % data.

One or more blood glucose results are excluded from the selected target % data because the test results are invalid.

 Results
Excluded

One or more
results may be
excluded from
low/high data.

One or more blood glucose results are excluded from the selected low/high data because the test results are invalid.

 Invalid Date


Check the date.

The date entered is not valid.
Enter the correct date.




A meter setting was changed while in Flight Mode.

The setting change will not take effect until Flight Mode is turned off.

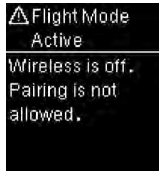


Pairing to a pump cannot be performed while in Flight Mode.

Retry pairing when the meter is not in Flight Mode.

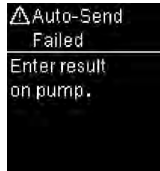

The blood glucose result has not been sent to the pump.

Manually enter the test result on the pump. For instructions, refer to the MiniMed™ System User Guide.

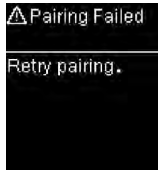

The meter was unable to pair with a pump.

Retry the pairing.

⊗ Strip Error

Reinsert or
replace with
new strip.

(E-1)

The test strip may be damaged, not properly inserted, or was previously used.

Remove and reinsert the test strip or replace it if damaged or previously used.

⊗ Test Error

Retest with
new strip.

(E-3)

A meter or test strip error has occurred.

Repeat the blood glucose test.

If a second E-3 code appears, perform a control test with the control solution and a new test strip.

- If the control result is within the acceptable range, review the proper testing procedure and repeat the blood glucose test with a new test strip.
- If the control result is not within the acceptable range, see the Understanding Out-of-Range Control Results section in the chapter Control Tests.

This error could appear if the cap on the test strip container was not closed tightly. The test strips may have been damaged due to improper storage or handling.

In rare cases, an E-3 error code may indicate that your blood glucose is extremely high and above the system's reading range. If you do not feel like your blood glucose is extremely high, repeat the blood glucose test. See the Unusual Blood Glucose Results section in the chapter Blood Glucose Tests. If the E-3 code still appears for your blood glucose test, **contact your healthcare professional immediately.**



Not enough blood or control solution was drawn into the test strip for measurement or was applied after the test had started.

Discard the test strip and repeat the blood glucose or control test.




Blood or control solution was applied to the test strip before **Apply drop** appeared.

Discard the test strip and repeat the blood glucose or control test.



An electronic error occurred.

Remove the batteries, press and hold the Power/Set/OK button for at least 2 seconds, and reinsert the batteries. Perform a blood glucose or control test.

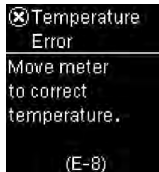

The temperature is above or below the proper range for the system.

Refer to the test strip package insert for system operating conditions. Move to an area with the appropriate conditions and repeat the blood glucose or control test. Do not artificially heat or cool the meter.



The batteries may be out of power.

Turn the meter back on. If the message continues to appear after several attempts, and the meter is not in a cold environment, replace the batteries. If the message reappears after the batteries have been replaced, remove the batteries, press and hold the Power/Set/OK button for at least 2 seconds, then reinsert the batteries.

⊗ Time/Date Error

Correct time/date if necessary.

(E-10)

The time and date setting may be incorrect.

Make sure the time and date are correct and adjust, if necessary.

⊗ Time/Date Error

Ensure pump is within range.

(E-10)

The time and date setting may be incorrect.

Make sure the pump is in communication range of the meter to update the time and date.

⊗ Test Error

Retest with new strip.

(E-11)

The test strip may be damaged. Retest with a new test strip.

⊗ High Ascorbate

Contact your healthcare professional.

(E-12)

Your blood sample may contain a high level of ascorbate.

Contact your healthcare professional.

⊗ Strip Error

Reinsert or replace with new strip.

(E-13)

Fluid or foreign material may be present in the test strip slot.

Remove and reinsert the test strip or repeat the blood glucose or control test with a new test strip. If the problem persists, contact Roche.

⊗ Electronic
Error
Contact Roche.

(E-14)

An electronic error has occurred.
Contact Roche.

⊗

(E-14)

Sync Time

Meter time has
been updated to
match time on
pump.

The meter time and date have
been changed to match the
paired pump.

Time/Date


Time and date
cannot be
changed while
paired to a
pump.

When paired to a pump, the
meter time and date cannot be
changed using the meter.

Logbook

No stored results

There are no results in the
Logbook.

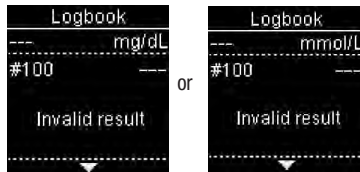

There is an invalid result in the Logbook.



There are no test results in range for the selected data.



Target Ranges is **Off** and there are no results for the Target % data stored in the meter.

Low/High Data

No data available.
Enable target
ranges to track
low/high BG data.

Target Ranges is **Off** and there are no results for the low/high data stored in the meter.

↑ High BG Data

No high BG data
available

↓ Low BG Data

No low BG data
available

There are no results stored in the meter for the selected data.

Patterns

No active
patterns based
on last 7 days of
data.

Patterns is **On** but there are no active patterns stored in the meter.

Patterns

No data available.
Turn on Patterns
in Settings to
show patterns
for results marked

Patterns is **Off**.

Patterns

with comments.



Blood glucose may be higher than the measuring range of the system.

See the Unusual Blood Glucose Results section in the chapter Blood Glucose Tests.



Blood glucose may be lower than the measuring range of the system.

See the Unusual Blood Glucose Results section in the chapter Blood Glucose Tests.

Product Limitations

See the literature packaged with the test strips and control solutions for the latest information on product specifications and limitations.

Specifications	
Blood volume	Refer to the test strip package insert.
Sample type	
Measuring time	
Measuring range	
Test strip storage conditions	
System operating conditions	
Meter storage conditions	Temperature: -25–70 °C
Memory capacity	720 blood glucose results and 32 control results with time and date
Automatic off	90 seconds
Power supply	Two 3-volt lithium batteries (coin cell type CR2032)
Display	LCD
Dimensions	80 × 47 × 20 mm (LWH)
Weight	Approx. 40 g (with batteries)
Construction	Hand-held
Protection class	III
Meter type	The Accu-Chek Guide Link meter is suitable for continuous operation.
Control solution storage conditions	Refer to the control solution package insert.
Interfaces	Bluetooth low energy technology; USB: micro-B connector (functionality not available)
Radio frequency connectivity	Bluetooth low energy technology operating in the frequency band of 2.402 GHz to 2.480 GHz with a maximum transmitted power of 0 dBm (1 mW).

8 Technical Information

Electromagnetic Compatibility – The meter meets the electromagnetic requirements as per EN61326-2-6 / EN60601-1-2. Its electromagnetic emission is thus low. Interference on the other electrically-driven equipment is not anticipated.

Performance Analysis – Refer to the test strip package insert.

Test Principle – Refer to the test strip package insert.

Declaration of Conformity – Roche hereby declares that the Accu-Chek Guide Link blood glucose meter conforms with the basic requirements and other relevant regulations of the European Directive 2014/53/EU. The conformity declaration may be found at the following website:
<http://declarations.accu-chek.com>

Communication Protocol – The Accu-Chek Guide Link blood glucose meter has been tested and certified against the Bluetooth Core Specification version 4.0.

Product Safety Information

WARNING

- Strong electromagnetic fields may interfere with the proper operation of the meter. Do not use the meter close to sources of strong electromagnetic radiation.
- To avoid electrostatic discharge, do not use the meter in a very dry environment, especially one in which synthetic materials are present.

Discarding the Meter













WARNING





- During blood glucose testing, the meter itself may come into contact with blood. Used meters therefore carry a risk of infection. Before discarding the meter, remove the battery or batteries. Discard used meters according to the regulations applicable in your country. Contact the local council and authority for information about correct disposal.
- Keep new and used batteries away from children. See the warning in the Introduction of this User's Manual for additional information.
- The meter falls outside the scope of the European Directive 2012/19/EU – Directive on waste electrical and electronic equipment (WEEE).
- Roche conforms to the Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.
- Discard used batteries according to local environmental regulations.

8 | Technical Information

Explanation of Symbols

These symbols may appear on the packaging, on the type plate, and in the instructions for the Accu-Chek Guide Link meter.

	Consult instructions for use
	Biological Risks – used meters carry a risk of infection.
	Caution, refer to safety-related notes in the instructions for use accompanying this product.
	The compliance mark indicates that the product complies with the applicable standard and establishes a traceable link between the equipment and the manufacturer, importer or their agent responsible for compliance and for placing it on the Australian and New Zealand market.
	Temperature limitation (store at)
	Admissible humidity range
	Use by
	Manufacturer
	Catalogue number
	Batch code
	In vitro diagnostic medical device
	Global Trade Item Number

	Serial number
	This product fulfils the requirements of the European Directive 98/79/EC on in vitro diagnostic medical devices.
	3-volt coin cell type CR2032
	Keep new and used batteries away from children.

Additional Supplies

Test Strips: Accu-Chek Guide test strips

Control Solutions: Accu-Chek Guide control solutions

8 | Technical Information

Information for Healthcare Professionals

WARNING

Healthcare Professionals: Follow the infection control procedures appropriate for your facility. Refer to the test strip package insert for additional healthcare professional information.

Sample Handling

Always wear gloves when handling blood-contaminated items. Always adhere to the recognised procedures for handling objects that are potentially contaminated with human material. Follow the hygiene and safety policy of your laboratory or institution. Prepare the selected blood collection site per facility policy.

Refer to the test strip package insert for additional information regarding acceptable sample types, anticoagulants, and handling instructions.

Recommending Alternative Site Testing to Patients

Decisions about whether to recommend Alternative Site Testing (AST) should take into account the motivation and knowledge level of the patient and his or her ability to understand the considerations relative to diabetes and AST. If you are considering recommending AST for your patients, you need to understand that there is a potential for a significant difference between fingertip or palm test results and test results obtained from the forearm or upper arm. The difference in capillary bed concentration and blood perfusion throughout the body can lead to sample site-to-site differences in blood glucose results. These physiological effects vary between individuals and can vary within a single individual based upon his or her behaviour and relative physical condition.

Our studies involving alternative site testing of adults with diabetes show that most persons will find their glucose level changes more quickly in blood from the fingertip or palm than in blood from the forearm or upper arm. This is especially important when blood glucose levels are falling or rising rapidly. If your patient is used to making therapy decisions based upon fingertip or palm test results, he or she should consider the delay, or lag time, affecting the test results obtained with blood from the forearm or upper arm.

Guarantee

The statutory provisions on rights in consumer goods sales in the country of purchase shall apply.

A

alternative site testing 23, 96
averages 58

B

battery, changing 79
battery type 91
blood glucose results, unusual 24
blood glucose test, performing 14
blood glucose, warning 18
button, functions 7

C

comments, adding to results 19
control results, out-of-range 32
control solution 27
control test, performing 28

D

data transfer, wireless 74

G

guarantee 97

H

healthcare professionals 96
high blood glucose 25
hyperglycaemia 25
hypoglycaemia 25

L

logbook 56
low blood glucose 25
low/high data 61

M

maintenance, meter 79
meter, cleaning 80

P

patterns 65
product limitations 91
product safety information 93
product specifications 91

S

settings, meter 33
beeper 37
language 53
last result 51
patterns 41
post meal 48
reminders 45
target ranges 38
time/date 36
time format 54
supplies 95
symbols
blood glucose results 20
IVD 94
meter display 10

T

target % data 59
technical information 91
test strips 14

U

use by date 14, 27

W

wireless communication
auto-send 72
delete pairing 76
flight mode 71
overview 69
pairing 70

Australia

Roche Diabetes Care Australia Pty. Limited
Accu-Chek Enquiry Line: 1800 251 816
www.accu-chek.com.au

Central America and the Caribbean

Local contact information at:
www.accu-chekcac.com

Egypt/Jordan/Kuwait/Lebanon

Roche Diabetes Care Middle East FZCO
Tel: +971 (0) 4 805 2222
www.rochediabetescareme.com
or contact Roche local authorized
representative in your country

Hong Kong

Enquiry hotline: +852-2485 7512
(office hours)
www.accu-chek.com.hk

L Y R A ehf.

Hádegismóar 4
110 Reykjavík

Ísland

www.lyra.is

India

Accu-Chek Customer Helpline: 1800 120 6020
(Toll-free)
www.accu-chek.in

Indonesia

Customer Support and Service Centre
Hotline: 0-800-1-222-999
www.accu-chek.com

Israel

Customer Support:
Tel. 04-6175390
www.dyndiabetes.co.il

Republic of Kazakhstan

Roche Kazakhstan LLP
77, Kunaev street, 15 floor
BC "Parkview office Tower"
050000, Almaty

Philippines

Accu-Chek Customer Hotline:
Phone: (02) 7718 7575 / (02) 7718 7588 /
+63 917 8978000
www.accu-chek.com.ph

Singapore


Accu-Chek ExtraCare line: 6272 9200
www.accu-chek.com.sg

Roche Diabetes Care South Africa (Pty) Ltd.
Hertford Office Park, 90 Bekker Road
Vorna Valley, 1686

South Africa

Tel: +27 (11) 504 4600
Email: info@accu-chek.co.za
www.accu-chek.co.za



 Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-chek.com

Equipment Type: Lower Power Device
TRC/SS/2018/63

Complies with
IDA Standards
N4511-17

54611/SDPPI/2018

ICASA TA-2017/2714
APPROVED



NTC
Type Approved
ESD-1816643C

IVD CE 0123



ACCU-CHEK and ACCU-CHEK GUIDE are trademarks of Roche.

MiniMed is a registered trademark of Medtronic, Inc.

The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

All other product names and trademarks are the property of their respective owners.

© 2020 Roche Diabetes Care
09941355001-1020

