



NOW FEATURING A CHILD-RESISTANT BATTERY DOOR

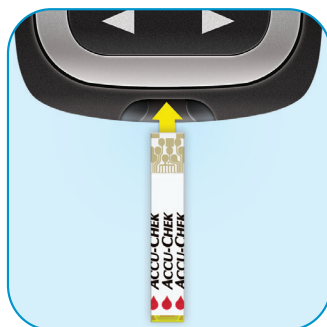

Test blood glucose

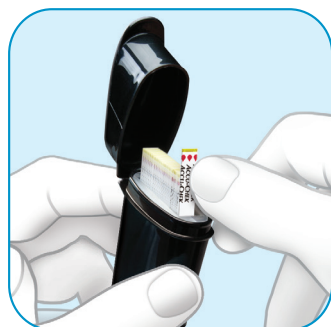

Remove a test strip from the spill-resistant container. Close the cap tightly.

Insert the metallic end of the test strip into the meter.

Touch the **yellow edge** of the test strip to the blood drop.

The test result appears on the display in less than 4 seconds.

Changing the Batteries

With more than 40 years of experience, our number one goal is to help persons with diabetes track and self-manage their blood glucose levels with high-quality products.

To support patient safety, Accu-Chek Guide Me meters now have a child resistant door.

Follow these simple instructions to change the batteries in your Accu-Chek Guide Me meter. Don't worry—your logbook data will be saved as you remove and replace the batteries.

1



To insert new batteries, open the child-resistant battery door, pushing the tab in the direction of the arrow and lift the battery door up.

2



Remove the old batteries.

3



Slide the new batteries under the black tabs with the (+) side facing up. Put the battery door back in place and snap it closed.

Note

- Check and remove any choke hazard stickers from the batteries prior to installing.
- The meter uses two 3-volt lithium batteries, coin cell type CR2032. It is a good idea to have spare batteries available.
- Always replace both batteries at the same time with the same high quality brand.

¹Brazg RL, et al. J. Diabetes Sci Technol. Nov 2016. DOI:10.1177/1932296816652902

WARNING – KEEP BATTERIES OUT OF REACH OF CHILDREN. If you suspect your child has swallowed or inserted a button battery immediately call the 24-hour Poisons Information Centre on 13 11 26 for fast, expert advice.

FOR PEOPLE WITH DIABETES. ALWAYS READ THE INSTRUCTIONS FOR USE. CONSULT YOUR HEALTHCARE PROFESSIONAL FOR ADVICE.

accu-chek.com.au | Accu-Chek Enquiry Line: 1800 251 816