


NOW FEATURING A CHILD-RESISTANT BATTERY DOOR




Test strip
Advanced accuracy for results you can trust.¹

Test blood glucose

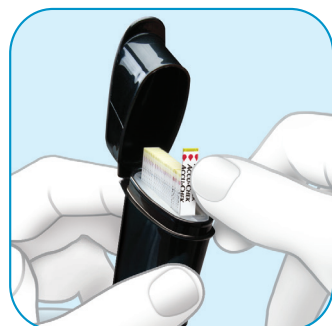

Remove a test strip from the spill-resistant container. Close the cap tightly.

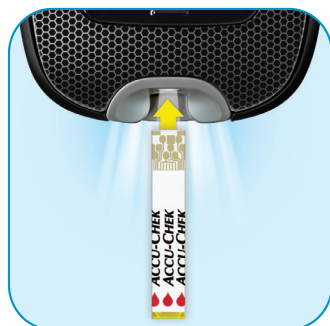

Insert the metallic end of the test strip into the meter.



Wait for the screen to show apply blood and then touch the **yellow edge** of the test strip to the blood drop.



The test result appears on the display in less than 4 seconds.

Changing the Batteries

With more than 40 years of experience, our number one goal is to help persons with diabetes track and self-manage their blood glucose levels with high-quality products.

To support patient safety, Accu-Chek Guide meters now have a child resistant door.

Follow these simple instructions to change the batteries in your Accu-Chek Guide meter. Don't worry—your logbook data will be saved as you remove and replace the batteries.

1



To insert new batteries, open the child-resistant battery door by inserting a narrow object into the slot, pushing the tab in the direction of the arrow, and lifting the door up.

2



Release the old batteries by pressing the  button. Remove the old batteries.

3



Slide the new batteries under the black tabs and  button, with the **(+)** side facing up. Put the battery door back in place and snap it closed.

Note

- Check and remove any choke hazard stickers from the batteries prior to installing.
- The meter uses two 3-volt lithium batteries, coin cell type CR2032. It is a good idea to have spare batteries available.
- Always replace both batteries at the same time with the same high quality brand.

¹Brazg RL, et al. J. Diabetes Sci Technol. Nov 2016. DOI:10.1177/1932296816652902

WARNING – KEEP BATTERIES OUT OF REACH OF CHILDREN. If you suspect your child has swallowed or inserted a button battery immediately call the 24-hour Poisons Information Centre on 13 11 26 for fast, expert advice.

FOR PEOPLE WITH DIABETES. ALWAYS READ THE INSTRUCTIONS FOR USE. CONSULT YOUR HEALTHCARE PROFESSIONAL FOR ADVICE.

accu-chek.com.au | Accu-Chek Enquiry Line: 1800 251 816

ACCU-CHEK and ACCU-CHEK GUIDE are trademarks of Roche. © 2020 Roche Diabetes Care. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Roche is under license. All other product names and trademarks are the property of their respective owners. Roche Diabetes Care Australia Pty Ltd, 24-32 Lexington Drive, Bella Vista NSW 2153, Australia. ABN 69 602 140 278. Date of preparation: August 2020. AU-688