

Accu-Chek® Lancing Systems

Less pain.¹ ***More gain!***

Get all the **Gains** from the world's first
1-Click lancing system with a **drum**.²



Smooth gentle lancing motion with Clixmotion®



No exposed lancets - less risk of accidental injury



Less things to carry - 6 lancets in each drum



Only as deep as you need - adjustable to 11 depth settings



Avoid accidental contamination - once removed, a drum cannot be reinserted

Accu-Chek® Lancing Systems

Less pain. **More gain!**

5 Tips for gentle blood sampling.

The FastClix Drum makes the Fastclix Safe to Handle, Gentle to Use.

As lancets penetrate the skin, it's important that you can lance quickly, comfortably and hygienically. All new and unused Accu-Chek lancets are sterile. With the Accu-Chek lancing system's **preloaded drum**, it's easy to dispose of used lancets.



1
Ensure hands are **washed and clean**.



2
Lance on the **side of the fingertip** rather than the pad.



3
Select a penetration depth set as shallow as possible but on a depth that still produces blood.



4
Keep the skin taut by pressing the lancing device firmly against the fingertip.



5
Alternate fingers **daily** and take steps to ensure **good blood circulation**.



References: 1. *Kocher, S, et al., Comparison of Lancing Devices for Self-Monitoring of Blood Glucose Regarding Lancing Pain J Diabetes Sci Technol 2009; 3(5):1135-1143.*
2. Lancing Comparison. Data on File.

FOR PEOPLE WITH DIABETES. ALWAYS READ THE INSTRUCTIONS FOR USE.
CONSULT YOUR HEALTHCARE PROFESSIONAL FOR ADVICE.

Accu-Chek lancing devices are for single patient use only. The same device must not be used by multiple patients. CLIXMOTION, ACCU-CHEK and FASTCLIX are trademarks of Roche. All other trademarks are the property of their respective owners.

©2020 Roche Diabetes Care. ABN 69 602 140 278. 09249222001.

AU-578. RDC6930. Date of preparation January 2020.