

ACCU-CHEK® Safe-T-Pro Plus

Sterile, single-use lancing device with 3 depth settings.



1. Remove the sterile cap

- Twist off the purple sterile cap.
- **Do not use the device if the purple sterile cap has been previously removed, as sterility may be compromised.**
- A built-in safety mechanism ensures that the lancing device can only be used once to prevent accidental needle stick injury.¹



2. Select penetration depth

- Accu-Chek Safe-T-Pro Plus offers 3 adjustable depth settings for different sample sizes and skin types.
- Select the desired penetration depth only after removing the sterile cap and choose the depth you require by turning the depth adjuster.*
- Depth settings:
 - 1. Low depth (1.3mm)
 - 2. Medium depth (1.8mm)
 - 3. High depth (2.3mm)



3. Obtain blood

- Press the Accu-Chek Safe-T-Pro Plus lancing device firmly against the puncture site.
- Push and release button down.[^]
- Collect blood.

[^]Trigger button resistance - to prevent accidental needle fire, firm pressure is required for needle to penetrate skin.



Learn how to use Accu-Chek meters, lancing devices and diabetes management tools at accu-chek.com.au and www.youtube.com/accucheck

www.accu-chek.com.au • Accu-Chek Enquiry Line: 1800 251 816

FOR PEOPLE WITH DIABETES. ALWAYS READ THE INSTRUCTIONS FOR USE. CONSULT YOUR HEALTHCARE PROFESSIONAL FOR ADVICE.

Accu-Chek lancing devices are for single patient use only. The same device must not be used on multiple patients.

*Please check your local guideline for penetration depth settings.

Reference: 1. Data on file.

ACCU-CHEK and SAFE-T-PRO are trademarks of Roche. ©2019 Roche Diabetes Care. All other trademarks are the property of their respective owners. Roche Diabetes Care Australia Pty Limited, 24-32 Lexington Drive, Bella Vista NSW 2153 Australia, ABN 69 602 140 278. 05691591001. AU-426. Date of preparation March 2019.